

INTENDED USE

This is a test to determine the presence of HIV antibodies in Human blood. Applicable to all groups including pregnant and neonates. This is a screening test only and all positive results must be followed up with an alternate test and counseling. Certain medical conditions may impact on the accuracy of the test, and should be queried with your physician.


MATERIALS AND REAGENTS SUPPLIED FOR TEST

Individually packed test cards, Plastic dropper, Sample diluent, Safety lancet, Alcohol swab, Package insert

WARNINGS AND PRECAUTIONS


- 1 **All positive results must be confirmed with by alternate method**
- 2 Treat all specimens as potentially infectious, wear gloves when handling specimens
- 3 Dispose of kit properly after use.
- 4 Do not use kit materials beyond the expiry date
- 5 Do not interchange reagents from different kits
- 6 Do not re-use test strips or any single use accessories

STEP BY STEP

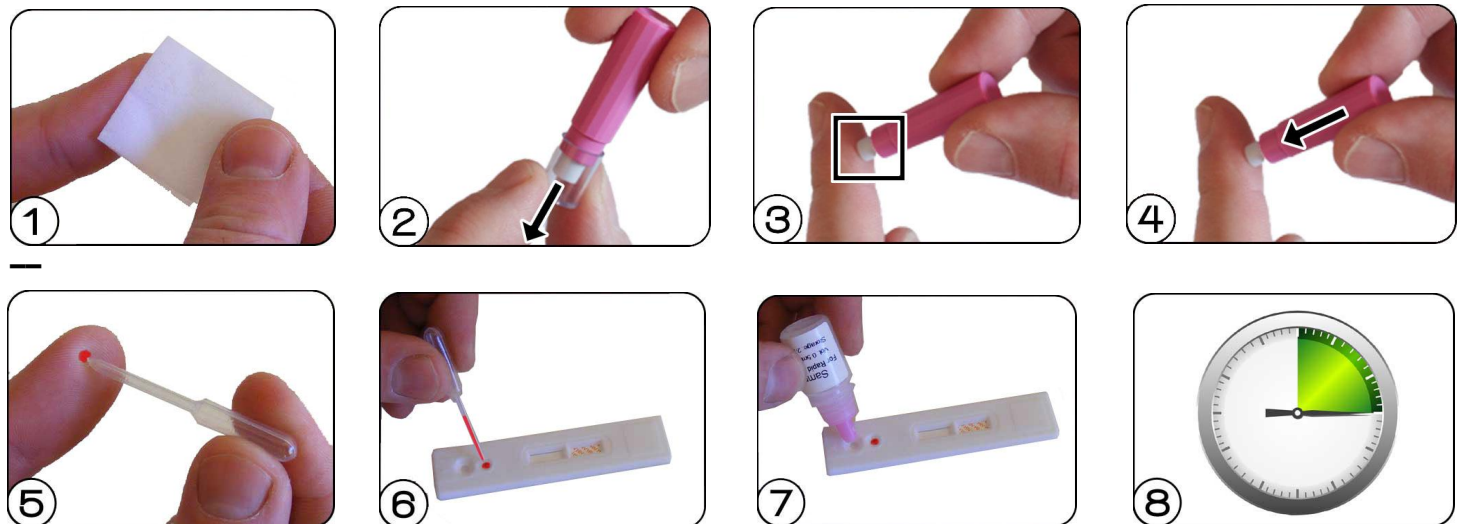

- 1 Bring all components to room temperature. Clean finger with alcohol swab
- 2 Remove cap from the safety lancet
- 3 Place white end of lancet against finger
- 4 Press the lancet firmly against finger until a click is heard
- 5 Draw up blood into plastic pipette
- 6 Deliver one drop of blood into the well marked "S"
- 7 Add 2 drops of the sample diluent into the well marked "D"
- 8 Wait for 15 minutes to interpret the results

INTERPRETING THE RESULTS

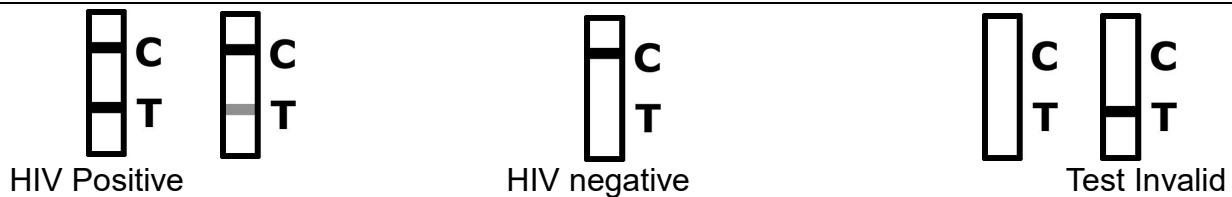

HIV Positive - A purple coloured line will appear next to C and T, it may be a bit lighter in T based on the number of antibodies present

HIV Negative - A purple coloured line will appear next to C only

Test Invalid - No lines will appear or the purple line will appear next to T only, the test is invalid you will need to use a new kit and retest.

Counseling is advised for both positive and negative test results, pre-test counseling is also recommended. More information on www.hivtestkits.co.za

National AIDS HELPLINE 0800 012 322

National HIV Health Care Workers Hotline 0800 212 506

Order HIV test kits from www.hivtestkits.co.za or telephonically on **011 482 5135 (South Africa)**